ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The user should be able to:

- Read and understand the directions, warnings and cautions.

INTENDED USE/INDICATIONS: Postoperative rotator cuff repairs, Postoperative Bankart procedures, Capsular shifts, Glenohumeral, dislocation/subluxation, Soft tissue strains/repairs

CONTRAINDICATIONS: None.

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Detach shoulder strap and open front panel. Position elbow in the sling as far back as possible. Loop the sling strap through the D-ring and secure. (Fig. 1)
2. Secure strap at the top of the sling. If the thumb strap is desired, attach strap at the front of the sling. (Fig. 1)
3. Place shoulder strap over the opposite shoulder. Connect shoulder strap to the sling using the quick release buckle. (Fig. 2)
4. Place the large pillow under the affected area with the 45° angle facing up. Attach the sling to the pillow, align the hook and loop strips. Wrap waist strap around the waistline and pillow and attach.

Note: Pillow may be turned upside down to create a 60° abduction angle.

CLEANING INSTRUCTIONS:

Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

The waist and shoulder strap may be trimmed by removing the Y-tab (Ultrasling II/III only).

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**RX ONLY.
INTENDED FOR SINGLE PATIENT USE.
NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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